


MINIMED™
INFUSION SETS
DESIGNED TO
SUIT EVERY
LIFESTYLE

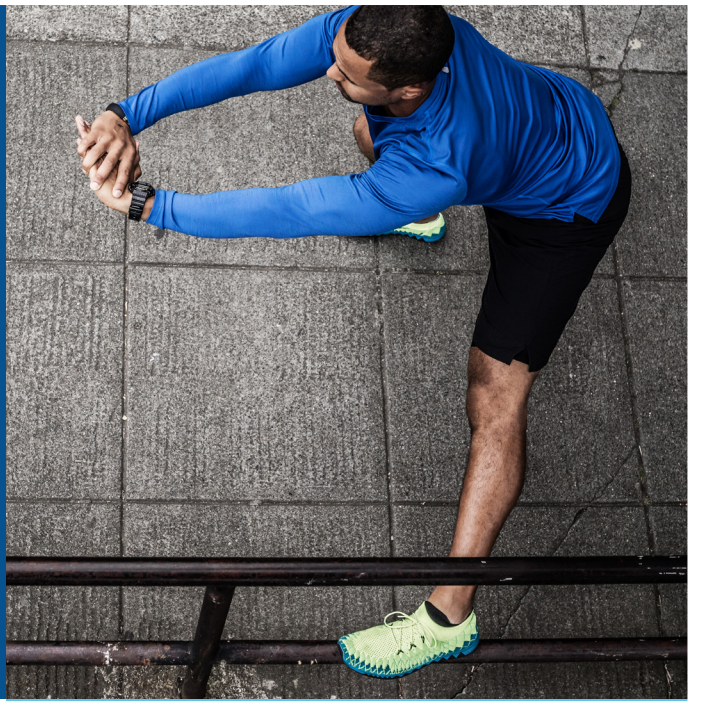

WELCOME

THIS GUIDE WILL INTRODUCE YOU TO OUR RANGE OF MINIMED™ INFUSION SETS.

MINIMED™ INFUSION SETS

Medtronic offers a wide range of infusion sets for you to choose the right infusion set for your **comfort and safety**.

The Infusion Set selection should be done with care and based on your **lifestyle, age** and **body build** also international clinical recommendations suggest.



Infusion sets and reservoirs management is an important part of your therapy management. The information here is intended as a guide. You should check with your healthcare professional about what may be best for you.



	SOFT CANNULA		PUMPS	SOFT CANNULA	
	MINIMED™ MIO™	MINIMED™ MIO™ 30	MINIMED™ SURE-T™	MINIMED™ QUICK-SET™	MINIMED™ SILHOUETTE
CANNULA/NEEDLE INSERTION ANGLE	90°	ANGLED	90°	90°	ANGLED
INSERTION DEVICE	ALL IN ONE	ALL IN ONE	MANUAL	SERTER	MANUAL/SERTER
NEEDLE/CANNULA LENGTHS	6mm 9mm	13mm	6mm 8mm 10mm	6mm 9mm	13mm 17mm
TUBING LENGTHS	45-60-80cm	60-110cm	45-60-80cm	45-60-80-110cm	45-60-80-110cm

For listing of indications, contraindications, precautions, warnings, and potential adverse events, please refer to the Instructions for Use.

MINIMED™ INFUSION SETS FIT FOR ANY LIFESTYLE

MINIMED™ MIO™

90° INSERTION
SOFT CANNULA
ALL-IN-ONE



Why choose the MiniMed™ Mio™?

MiniMed™ Mio™ is an all-in-one infusion set, integrating the set and the insertion device in one single unit that comes in a variety of colours.

The MiniMed™ Mio™ is easy to insert and carry, making it a great choice for anyone looking for a portable and reliable infusion set, offering convenience whilst on the go.

- Convenient - Integrated 'all-in-one' infusion set and insertion device
- Practical - Rigid packaging
- Easy at-site disconnection - Press and pull motion
- 3 colours to choose from (Blue, Pink and Clear)

The Mio™ infusion set is particularly suitable for:

- Both children and adults of average to heavier build
- Patients who have dexterity or vision problems
- Patients preferring to insert the infusion set in areas that are harder to reach (alternative sites)

ALL-IN-ONE PERSONALISATION



MINIMED™ MIO™ 30

30° INSERTION
SOFT CANNULA
ALL-IN-ONE



ALL-IN-ONE
CONVENIENCE

MINIMED™ SURE-T™

90° INSERTION
STEEL NEEDLE



Why choose the MiniMed™ Mio™?

MiniMed™ Mio™ 30 is the newest innovation in the MiniMed™ infusion sets portfolio.

Responding to the patients' needs, MiniMed™ Mio™ 30 improves flexibility, offering angled insertion and an all-in-one infusion set with integrated inserter, for better convenience whilst on the go.

- Convenient - Integrated 'all-in-one' infusion set and insertion device
- Practical - Rigid packaging
- At-a-glance site inspection made possible with the clear window on the set
- Easy at-site disconnection - Press and pull motion

The MiniMed™ Mio™ 30 infusion set is particularly suitable for:

- Lean or muscular children or adults
- Patients with recurrent site infections
- Patients with lipohypertrophy and/or dexterity problems

SECURITY,
SIMPLY



Why choose the MiniMed™ Sure-T™?

MiniMed™ Sure-T™ is uniquely designed to provide continuous insulin delivery. Its kink-free steel needle; the thinnest available in the MiniMed™ portfolio, combined with an extra disconnection site 10 cm away from the infusion site, provides extra security against dislodging.

- Convenience for children - Tubing connector 10 cm away from insertion site allows disconnection without touching it (press and pull motion)
- Additional adhesive pad for extra security and reduced risk of needle pull-out
- Comfort - 29-gauge steel needle is the thinnest available from Medtronic
- Manual insertion

MiniMed™ Sure-T™ is particularly suitable for:

- Children of normal to heavier build
- Pregnant women
- Patients who are allergic to Teflon® and have reactions to plastic cannulae,
- Patients who are fit and active with a history of bent cannulae
- Patients or parents and caregivers who are hesitant about complex infusion site changes

MINIMED™ QUICK-SET™ 30


90° INSERTION
SOFT CANNULA

COMFORT
AND EASE



MINIMED™ SILHOUETTE™

VARIABLE ANGLE INSERTION
SOFT CANNULA

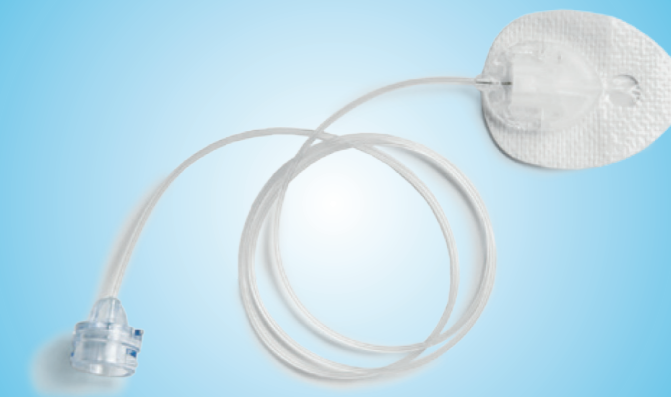

Why choose the MiniMed™ Quick-set™?

MiniMed™ Quick-set™ is a popular MiniMed™ infusion set combining ease of use with comfort.

The MiniMed™ Quick-serter insertion device is a spring-activated device designed to make Quick-set™ insertions easy with the press of a button, even in hard-to-reach areas.

- Consistent insertion with optional MiniMed™ Quick-serter™. Manual insertion is also possible, if preferred
- Easy at-site disconnection - Press, turn and lift motion
- Reduce risk of needlestick injury with fold-over needle guard

The MiniMed™ Quick-set™ infusion set is particularly suitable for:

- Both children and adults of average to heavier build
- Patients who need simplicity in set insertion
- Children learning to insert their own sets
- Patients with poor dexterity

COMFORT AND FLEXIBILITY




Why choose the MiniMed™ Silhouette™ ?

MiniMed™ Silhouette™ offers flexibility, with a variable insertion angle from 20 to 45° enabling you to adjust the depth of insertion.

- Controlled insertion with optional MiniMed™ Sil-serter™. Manual insertion is also possible, if preferred.
- At-a-glance site inspection made possible with the clear window on the set
- Flexibility - Adjustable angle insertion allows you to choose
- Easy at-site disconnection - Press and pull motion

MiniMed™ Silhouette™ is particularly suitable for:

- Lean or muscular children or adults
- Patients with a higher risk of the set being pulled out (including active children and sporty individuals)
- Patients with recurrent site infections
- Patients with lipohypertrophy

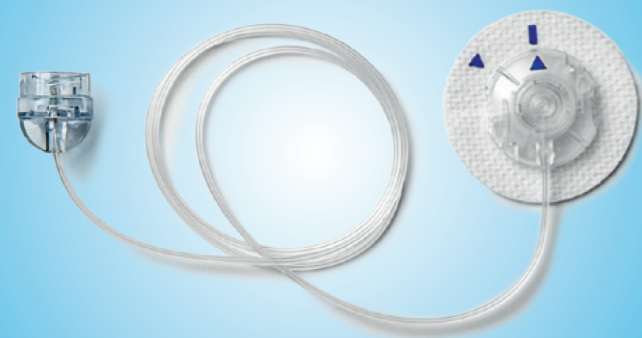


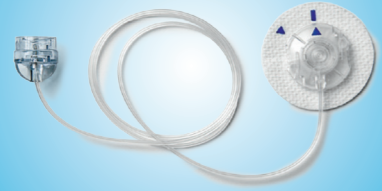
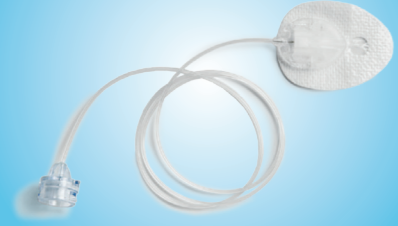


MiniMed™
Quick-serter™
insertion device
for MiniMed™
Quick-set™
infusion set

MiniMed™
Sil-serter™
insertion device
for MiniMed™
Silhouette™
infusion set



INFUSION SETS & RESERVOIRS

PRODUCT NAME	LENGTH		MEDTRONIC CODE (BY CONNECTION TYPE)		
	Cannula/ Needle (mm)	Tubing (cm)	MiniMed™ connector (for Paradigm 511, 512/712, 515/715, 522/722, Veo & MM640G pumps)	Luer-lock connector (for 507/507c, 508 pumps) (Note, products not described in this guide)	
MINIMED™ MIO™ INFUSION SET (10 PER BOX) 	PINK	6	45	MMT-921	-
		6	60	MMT-923	-
		6	80	MMT-925	-
	BLUE	6	45	MMT-941	-
		6	60	MMT-943	-
		6	80	MMT-945	-
	CLEAR	6	80	MMT-965	-
		9	80	MMT-975	-
	MINIMED™ MIO™ 30 INFUSION SET (10 PER BOX) 	13	60	MMT-905	-
13		100	MMT-906	-	
MINIMED™ SURE-T™ INFUSION SET (10 PER BOX) 	6	60	MMT-864	-	
	6	80	MMT-866	-	
	8	60	MMT-874	-	
	8	80	MMT-876	-	
	10	60	MMT-884	-	
	10	80	MMT-886	-	
	6	60	-	MMT-863	
	6	80	-	MMT-865	
	8	60	-	MMT-873	
	8	80	-	MMT-875	
	10	60	-	MMT-883	
	10	80	-	MMT-885	
	MINIMED™ QUICK-SET™ INFUSION SET (10 PER BOX) 	6	60	MMT-399	-
		6	80	MMT-387	-
6		110	MMT-398	-	
9		60	MMT-397	-	
9		80	MMT-386	-	
9		110	MMT-396	-	
6		60	-	MMT-393	
6		110	-	MMT-391	
9		60	-	MMT-392	
9		110	-	MMT-390	
MINIMED™ SILHOUETTE™ INFUSION SET (10 PER BOX) 	13	45	MMT-368	-	
	13	60	MMT-381	-	
	13	80	MMT-383	-	
	13	110	MMT-382	-	
	17	60	MMT-378	-	
	17	80	MMT-384	-	
	17	110	MMT-377	-	
	17	60	-	MMT-373	
	17	110	-	MMT-371	
	MINIMED™ INSULIN RESERVOIR (10 PER BOX)	1.8ml	-	-	MMT-326A
3.0ml		-	-	MMT-332A	-
MINIMED™ QUICK-SERTER™ INSERTION DEVICE			MMT-305QS		
MINIMED™ SIL-SERTER™ INSERTION DEVICE			MMI-385		

INTRODUCING THE MINIMED 640G SYSTEM WITH EXCLUSIVE SMARTGUARD TECHNOLOGY INTELLIGENT FOR BETTER CONTROL



INSULIN DELIVERY

Benefit from the unique MiniMed™ connection for insulin delivery, with an infusion set for every lifestyle.



CONTINUOUS GLUCOSE MONITORING

Continuously monitor your glucose levels with superior comfort* and accuracy**, and receive an alert when you are at risk of going too high or too low.



THERAPY MANAGEMENT TOOLS

Upload your pump to CareLink™ Pro software to conveniently track your glucose control and remotely share this information with your healthcare professional.



BLOOD GLUCOSE TESTING

CONTOUR™ NEXT LINK 2.4 meter links wirelessly to MiniMed 640G and provides proven testing accuracy*** and discreet remote bolusing.

*The size of the new generation Enlite sensor has been reduced by 80%. **When combined with the MiniMed 640G system and Guardian 2 link transmitter. Enlite Sensor Performance addendum to user guide. ***Section 8 clinical study. Data on file. Bayer Healthcare, LLC.

REPUBLIC OF IRELAND

Medtronic Ireland Limited

Cherrywood Business Park, Block G,
Cherrywood Science and Technology Park,
Loughinstown, Co. Dublin

www.medtronic-diabetes.co.uk

UNITED KINGDOM

Medtronic Limited

Building 9, Croxley Park, Hatters Lane,
Watford, Herts. WD18 8WW
Tel: 44 (0)1923 205 167

Medtronic

Safety Information: MiniMed™ 640G Insulin Pump is indicated for the continuous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. In addition, the Enlite™ glucose sensor is indicated for continuous or periodic monitoring of glucose levels in the fluid under the skin, and possible low and high blood glucose episodes. The pump displays continuous glucose values and stores this data so that it can be analysed to track patterns and improve diabetes management. Pump history can be downloaded to a computer for analysis of historical glucose values. The continuous glucose values provided by the MiniMed™ 640G insulin pump are not intended to be used directly for making therapy adjustments. Rather, they provide an indication that a confirmation fingerstick measurement may be required. All therapy adjustments should be based on measurements obtained using a home glucose monitor and not based on the value displayed by the pump.

Safety Information: Medtronic CareLink® Software is intended for use as a tool to help manage diabetes. The purpose of the software is to take information transmitted from insulin pumps, glucose meters and continuous glucose monitoring systems, and turn it into Medtronic CareLink® reports that can be used to identify trends and track daily activities—such as carbohydrates consumed, meal times, insulin delivery, and glucose readings. Medtronic CareLink® report data is intended for use as an adjunct in the management of diabetes only and NOT intended to be relied upon by itself. Patients should consult their healthcare professionals familiar with the management of diabetes prior to making changes in treatment. Medtronic, MiniMed, Mio, Sure-T, Quick-set, Silhouette, CareLink, Quick-setter and Sil-setter are trademarks of Medtronic MiniMed, Inc. The legal manufacturer of MiniMed™ Mio™, MiniMed™ Mio™ 30, MiniMed™ Quick-set™ MiniMed™ Sure-T™ and MiniMed™ Silhouette™ is: Unomedical a/s, Aaholmvej 1-3, Osted, 4320 Lejre, Denmark.

©2015 Medtronic International Trading Sarl. All Rights Reserved. No part of this brochure may be reproduced or utilised in any form or by any means without the written permission of Medtronic International Trading Sarl. Printed in Europe.

Please visit www.medtronicdiabetes.com/important-safety-information for complete details.

UC201710515 EN Date of prep: April 2017