GUARDIAN[™] CONNECT

QUICK START GUIDE GUIDE DE DÉMARRAGE KURZANLEITUNG GUÍA DE INICIO RÁPIDO BEKNOPTE HANDLEIDING GUIDA DI AVVIO RAPIDO

Congratulations on taking the next step in managing your diabetes! This Quick Start Guide outlines the key steps to start using your Guardian Connect system.

Please consult the User Guides in your Starter Kit for full product information, including important warnings and limitations of use.

BEFORE GETTING STARTED

1 Register your device online at

www.guardianconnect.medtronic-diabetes.com

Access self-guided training, and find other helpful resources before you get started.

2 Charge your Guardian Connect transmitter – It can take up to 2 hours

Place the transmitter on the charger. The green light on the charger will flash. The transmitter is fully charged when the green light is off.

3 Download the Guardian Connect app from your mobile local device app store

GETTING STARTED

We advise you to start setting-up your Guardian Connect system in the morning.

1 Gather your supplies & turn on Internet connection and Bluetooth on your mobile

Open the Guardian Connect app and follow the Start-up Wizard with your sensor, serter, fully charged transmitter, overtape, wireless or mobile connection and Bluetooth turned on.

Make sure you always have Bluetooth turned on when using your Guardian Connect system.

a. Create your Carelink Personal account on https://carelink.minimed.eu

GOOD TO KNOW

3

မှာ

You can directly do so from your mobile device following the Wizard. Don't forget to scroll to the very top of the page to complete the verification. After clicking on the "I am not a robot" box, you will be required to tick on some images before completing your registration.

b. Pair your transmitter, following the steps on the screen.

c. Insert and start your sensor. Gather all of your supplies including your serter, sensor, sensor overtape and your transmitter. Refer to your serter user guide for instructions on how to insert the sensor. Once you have inserted and taped your sensor, connect your transmitter to the sensor and press "Start New Sensor" on your Guardian Connect app Wizard to complete the sensor start up.

d. Personalize your alerts. You will then be asked to set-up your alerts and thresholds. You will be able to modify those from the app later.

Calibrations are necessary to keep displaying sensor glucose readings and to ensure optimal sensor accuracy. To calibrate, you must take a BG (fingerstick) and **immediately** enter that value into your Guardian Connect app by pressing on the "Drop" icon \triangle .

The best time to calibrate is usually just before meals or before going to bed. Calibrations are key for system accuracy and will be requested by the app. Right after inserting a new sensor, on day 1:

- Within 2 hours after you connect the transmitter to your sensor and start the warm up period a "Calibrate Now" alert will occur. Following this first calibration your sensor readings will start displaying within 5 minutes.
- 3 or 6 hours after the first calibration, the second calibration will be due.
 To know by when this second calibration is due, tap the "Drop" icon.

After the first two start-up calibrations, calibrations are needed every **12 hours.**

GOOD TO KNOW

The Guardian Connect system monitors your sensor readings throughout the day to ensure your sensor glucose readings remain accurate. In some cases, the system may require you to calibrate the sensor more frequently if it detects the accuracy could be improved. "Calibrate Now" alerts may occur more frequently than every 12 hours to maintain the accuracy of your sensor. The "Drop" icon timing will be updated automatically.

(Optionally) Create accounts for your care partners, and contact them with this information for them to finish the set-up

Press the Menu icon on the top left corner of the Home Screen of the app. Tap on Sync to CareLink / Manage Care Partners. You are automatically redirected to the CareLink Connect tab on https://carelink.minimed.eu. Log-in with your CareLink credentials. Then tap the Menu button on the top left corner / Connect Settings / Add Care Partner.

 \odot

Fill-in the mandatory fields for your care partner (First, Last name). Create a username and temporary password for him, then click Save. This temporary password is valid for 24 hours. You are not yet connected to your care partner.

Now, you need to contact your care partner.

- Provide them with their temporary CareLink credentials.
- There is no need for your care partner to download the Guardian Connect App.

- Tell your care partner they have 24 hours to log-in to https://carelink.minimed.eu (from any Internet connected devices) to finish setting up their account.
- Your care partner will be requested to change their password upon logging in to https://carelink.minimed.eu for the first time.
- They will then be requested to submit their mobile phone number and select the SMS alerts they wish to receive.

Only after you have completed these steps, will you be connected and sharing your glucose information with your care partner.

GOOD TO KNOW

To use care partner alerts successfully:

- Your smartphone must be connected to WiFi or a mobile data network (e.g. a 3G or 4G network), and be running the Guardian Connect app.
- Your care partner must have an active CareLink Connect account, with the desired alerts enabled on the CareLink Connect settings screen.
- Your care partner should adjust the alert delay setting as needed. With no delay set, alerts will be forwarded to your care partner if you don't snooze or clear them immediately (within about 10-15 seconds).

SMS alerts can help care partners stay updated about your glucose levels. However, interruptions to your Internet connection or your care partner's mobile network could result in SMS alerts not being received. It is important you and your care partner do not rely exclusively on these alerts as part of your diabetes treatment plan.

For more information on the CareLink Connect tab and how to manage care partners, please refer to the CareLink Connect User Guide found in the CareLink Connect tab of CareLink Personal.

Information contained herein does not replace the User Guide. Please refer to the User Guide for more information.

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- EN ©2017 Medtronic MiniMed, Inc. All rights reserved. CareLink® and MiniMed® are registered trademarks of Medtronic MiniMed, Inc. Bluetooth® is a registered trademark of Bluetooth SIG, Inc.
UC201708355EE©2017 Medtronic. Date of preparation: December 2016. All rights reserved. Printed in Europe.