

# A SIMPLE CHANGE FOR THE BETTER



Medtronic

# MINIMED™ MIO™ ADVANCE SET CHANGES MADE EASY

## FEWER STEPS, FASTER INSERTION

**Fill** your reservoir as per your pump instructions.

**Prime** your infusion set by filling the tubing until you see drops appear.



**PEEL** ADHESIVE



**PINCH** AND REMOVE SAFETY PIN



**PLACE** ON BODY



**PRESS** THE BUTTON



**MINIMED™ MIO™ ADVANCE** requires a cannula fill of **0.6 units**

\* For quick reference only. For full instructions please refer to the IFU.  
For a training video and further details please visit:  
[www.medtronic-diabetes.co.uk](http://www.medtronic-diabetes.co.uk)

**United Kingdom**  
Medtronic Limited  
Building 9, Croxley Park  
Hatters Lane,  
Watford, Herts WD18 8WW  
Tel: +44 (0)1923 205 167  
Fax: +44 (0)1923 241 004  
[www.medtronic-diabetes.co.uk](http://www.medtronic-diabetes.co.uk)

**Medtronic Ireland Limited**  
Block 3090-3094,  
Lake Drive  
Citywest Business Campus,  
Dublin, DN24 XN47  
Tel: +353 (0)15111499  
Fax: +353 (0)18077220  
[www.medtronic-diabetes.ie](http://www.medtronic-diabetes.ie)

