


Getting started with the CareLink™ Connect app



1. Open the app and follow the on-screen instructions. You may be warned that your device is untested but can still continue.

2. Log into your CareLink™ Personal care partner account. Sign up if you do not already have one

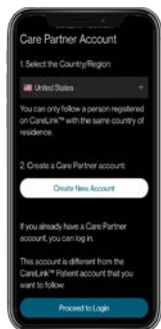


Note: Your country must be the same as the person you will be following

3. Read the Warning screen and accept by tapping Next



4. Enter the CareLink™ Personal username of the person you wish to follow*


5. Once accepted, you will be prompted to set up your notifications. You will not receive any notifications** until these are set

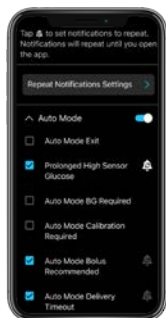

6. Tap **Follow another person** to add up to 5 MiniMed™ Mobile app users to your account



Each link will require notifications setting up

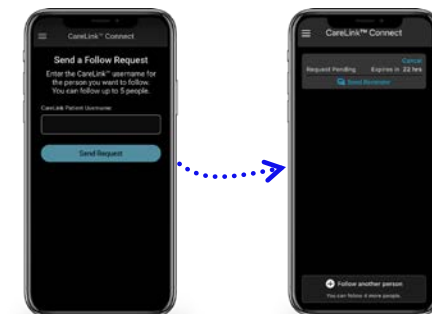

They must accept your connection request in their CareLink™ Personal account within 24hrs

Compatible phone? Check your device at:




TIP: For an improved experience, automatic updates to mobile operating systems should be deactivated - if one becomes available, check compatibility to see if it is supported before installing

Representative patient profile, Google Play is trademark of Google LLC and App Store is a trademark of Apple Inc.

*See local website for supported devices - must be using and logged into the MiniMed Mobile app

**Internet connection required for live data and notifications - Phone screens may look different depending on the pump system and mobile phone used.


Linking to multiple care partners

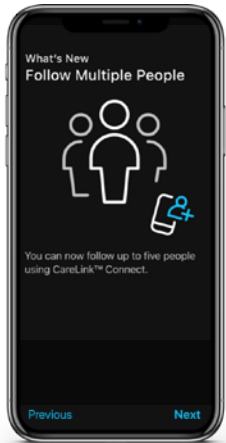

CareLink™ Connect app users can follow up to 5 MiniMed™ Mobile users in one app. You can switch between live views and receive push notifications from them all, as set by you.



Notifications

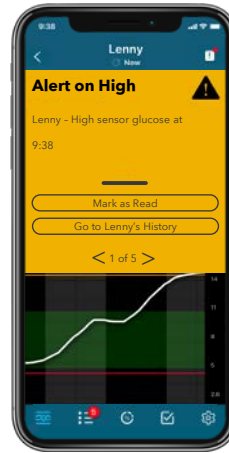

Notifications are colour coordinated:


- Alarms** are red
- Alerts** are yellow
- Reminders are grey
- Messages** are blue

Each care partner that you are following must have notifications set up individually

Important Information:


- Notifications must be set in the Settings menu before any will be received.
- Having your phone on **Do Not Disturb, Mute** or **Vibrate** may cause you to miss notifications
- The CareLink™ Connect app lets you follow up to 5 MiniMed™ Mobile users per account. There is a help section in the Main menu

Notification privacy

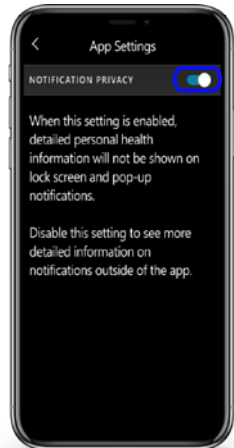


You can select if you want to see health related information in your notifications or have it hidden.

Privacy option switched on:

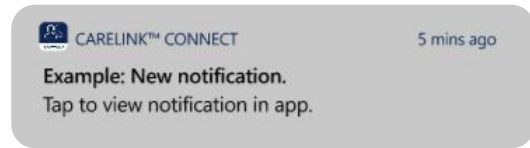

Or

Privacy option switched off:

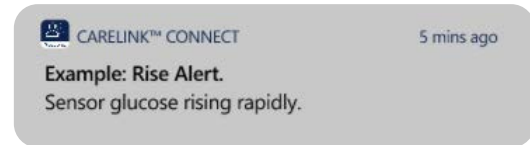

Repeat notifications:

- Some notifications can be set to repeat when received at certain times you specify if they have this symbol
- The notification sound can be personalised and will sound for 30 seconds and repeat every 60 seconds until the app is viewed

