


Each bolus event can contain up to 3 numbers. All 3 numbers do not have to be present to be recorded. Any number can be displayed by itself or all at the same time.

- Top number is your BG reading. In this example, the BG is 11.7 mmol/L
- The middle number in the black bar is your carb entry. In this example, 15 grams of carbs were consumed at this time.
- The bottom number is the insulin dose given. In this example, 2.3 units were given for this bolus event


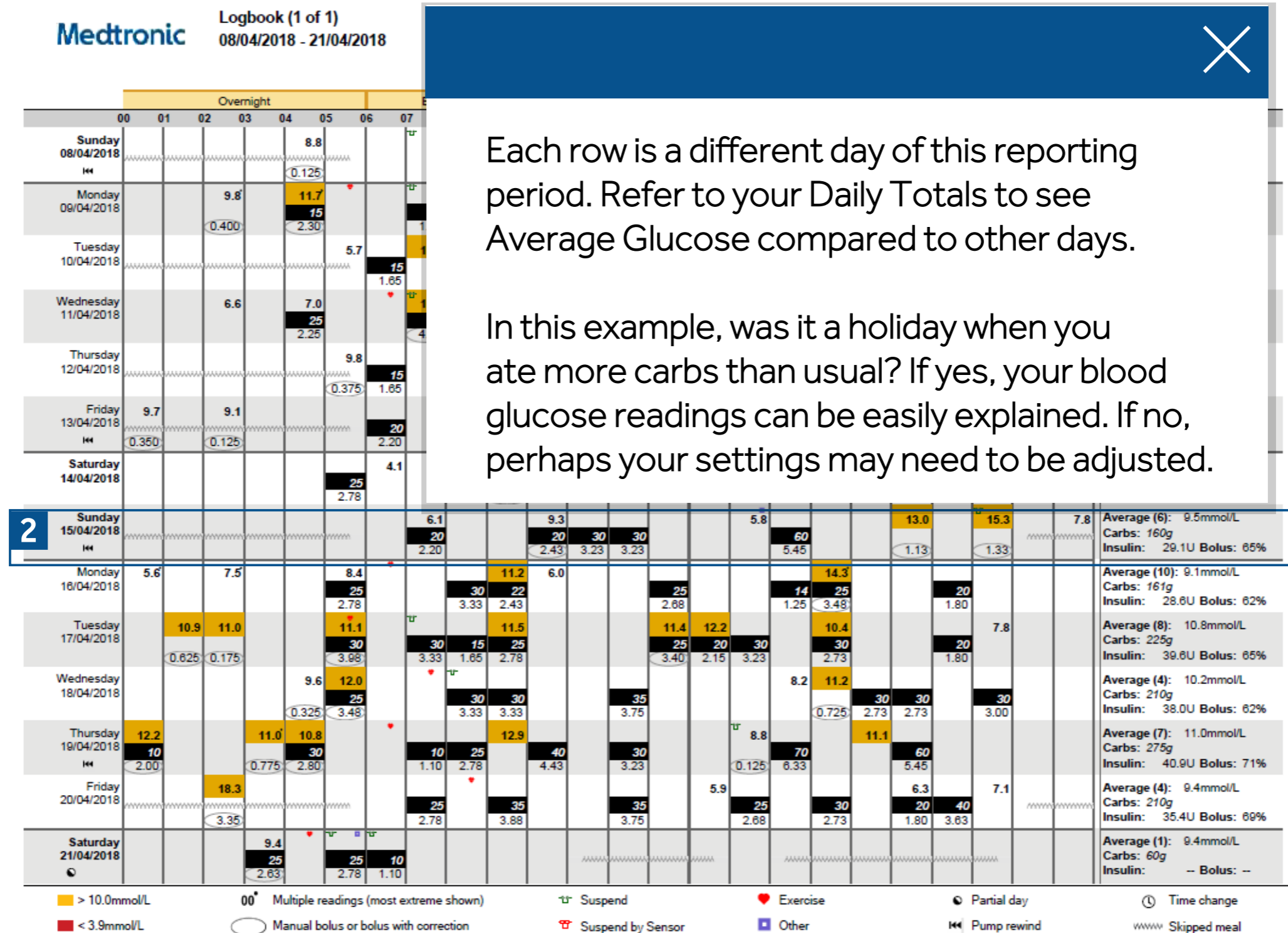


Each row is a different day of this reporting period. Refer to your Daily Totals to see Average Glucose compared to other days.

In this example, was it a holiday when you ate more carbs than usual? If yes, your blood glucose readings can be easily explained. If no, perhaps your settings may need to be adjusted.






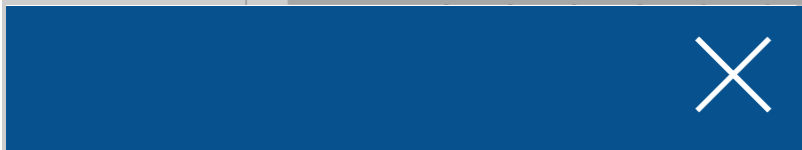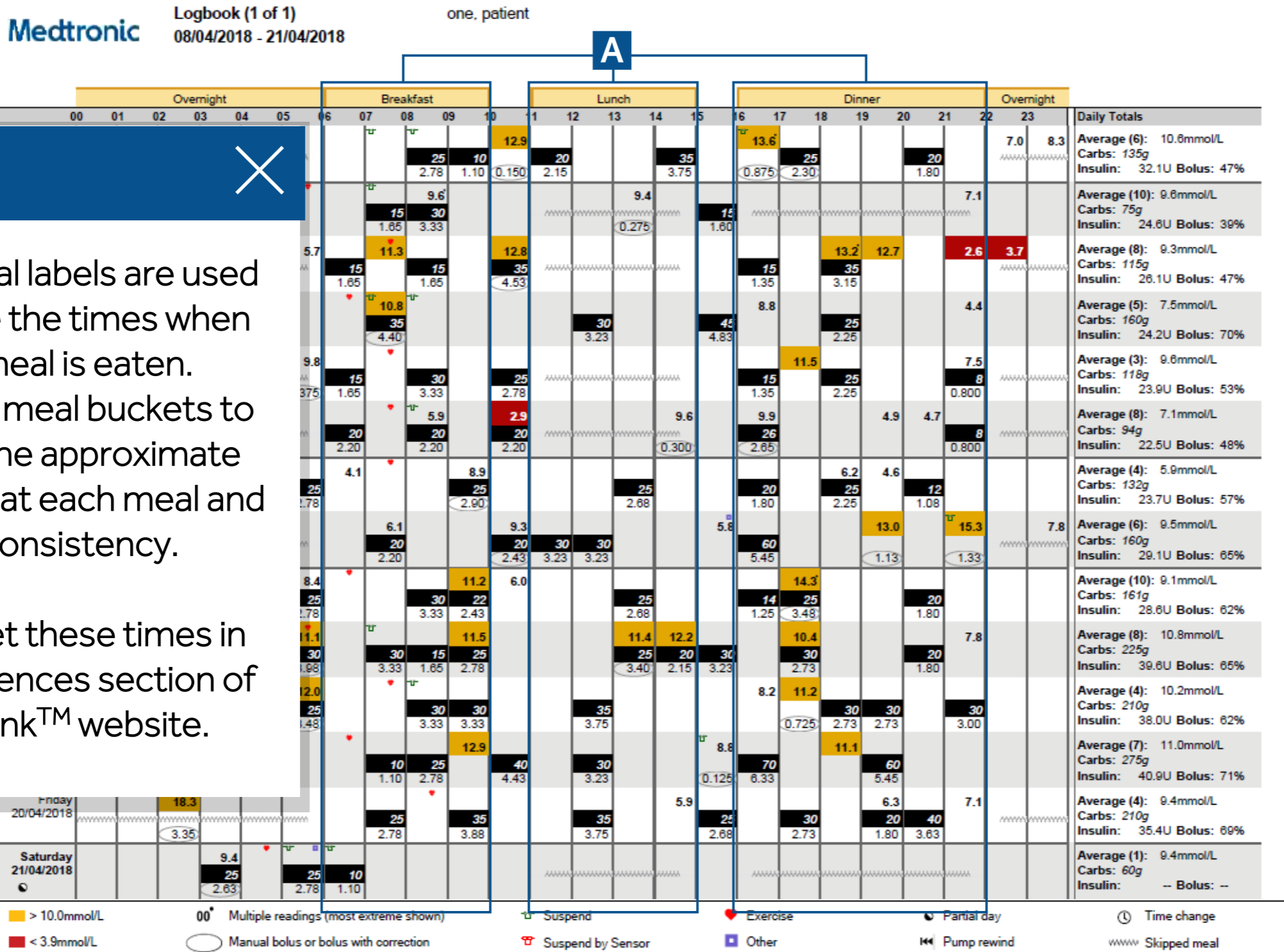

These meal labels are used to indicate the times when a certain meal is eaten. Use these meal buckets to organize the approximate time you eat each meal and note any consistency.

You can set these times in the Preferences section of the CareLink™ website.

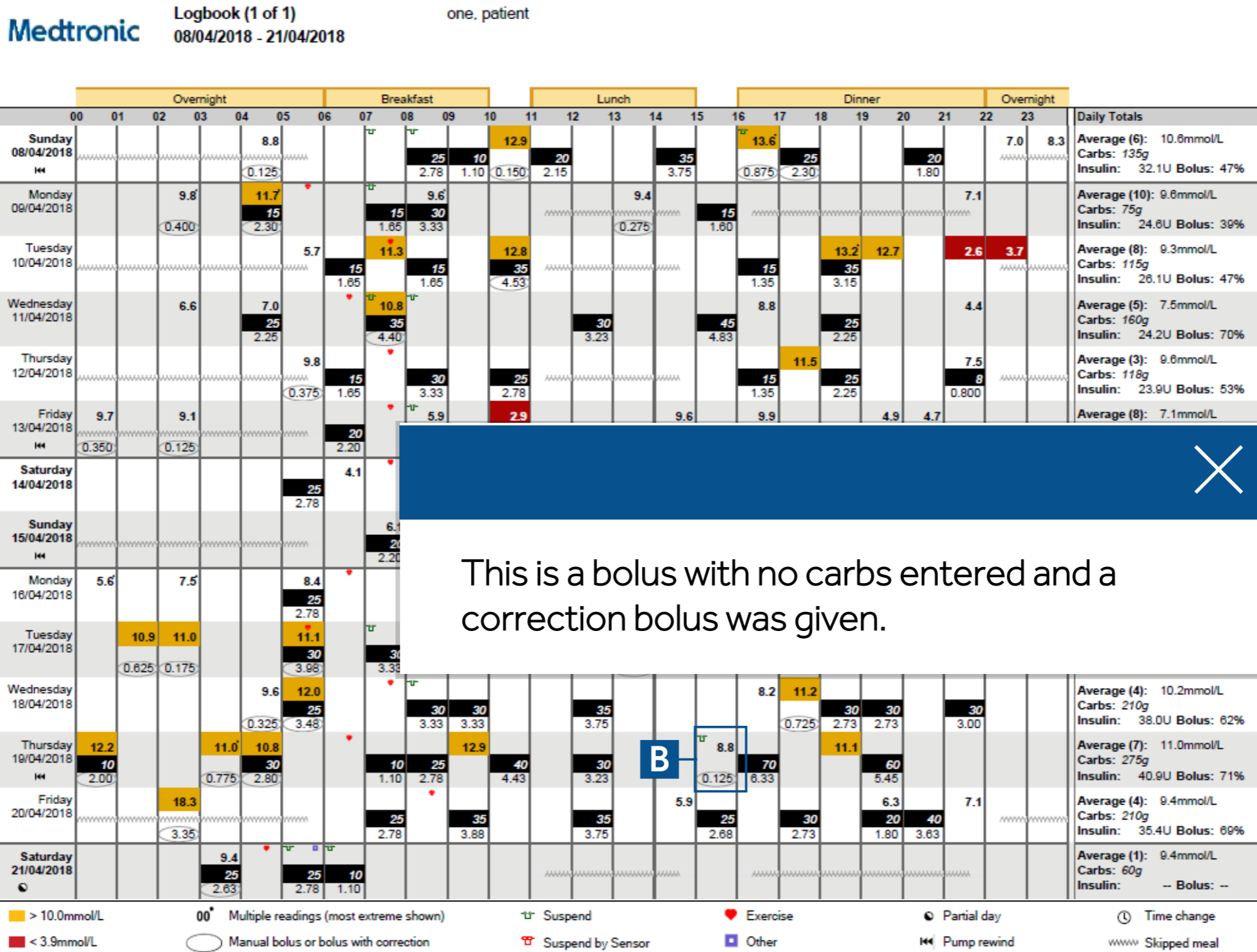

## Medtronic

EUROPE  
Medtronic International Trading Sàrl  
Route du Molliau 31  
Case postale  
CH-1131 Tolochenaz  
Switzerland  
Tel: +41 (0) 21 802 70 00  
Fax: +41 (0) 21 802 79 00  
[www.medtronic.eu](http://www.medtronic.eu)

### Safety Information: CareLink™ software

CareLink™ software is intended for use as a tool to help manage diabetes. The purpose of the software is to take information transmitted from insulin pumps, glucose meters and continuous glucose monitoring systems, and turn it into CareLink™ reports. The reports provide information that can be used to identify trends and track daily activities such as carbohydrates consumed, meal times, insulin delivery, and glucose readings. NOTE: CareLink™ report data is intended for use as an adjunct in the management of diabetes only and NOT intended to be relied upon by itself.

For a listing of indications, contraindications, precautions, warnings and potential adverse events please refer to the instructions for Use. UC201903975 EN

© 2018 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.